

SAPPHIRE[®] II PRO 1.0 mm - 1.5 mm

Coronary Dilatation Catheter

Cross with Confidence Every Time

Redefining the standard for crossing chronic total occlusions

Single-component **Sub-Zero tapered** tip, tailored for the smaller catheter size

Durable balloon material and **higher RBP** for resistant CTO lesions

1.0 mm
for Complex
CTO lesions

Increased inner body wall thickness to **reduce guidewire friction**

Single marker band and improved balloon folding for a **low crossing profile**

Spiral-cut hypotube transition zone to reduce kinking and maintain **excellent force transmission**


Technical Specifications

Proximal Shaft Diameter	1.9F	Tip Entry Profile	0.016"
Distal Shaft Diameter	2.36F	Marker Bands	1 marker band (centered)
Leading Tip Length	1.5 mm	Coating	Hydro-X (distal tip to guidewire exit port); Invio™ (guidewire lumen)
Balloon Material	Nylon	Crossing Profile	0.0216" (Ø 1.0 mm)
Balloon Pleats	2-fold (Ø 1.0 mm); 3-fold (Ø 1.25-1.5 mm)	Nominal Pressure	6 atm
Compliance	Semi-compliant	Rated Burst Pressure	16 atm


SAPPHIRE[®] II PRO 1.0 mm - 1.5 mm

Coronary Dilatation Catheter

Tip Entry Comparison*

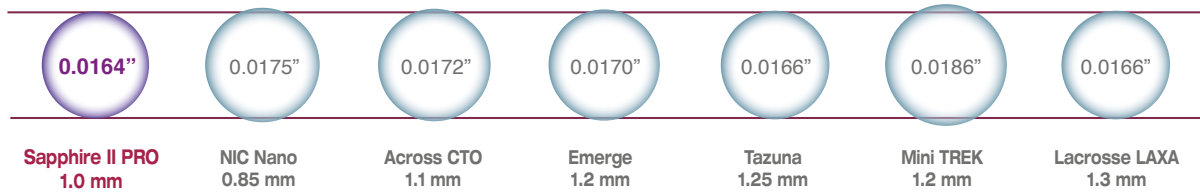

Crossing Profile Comparison*

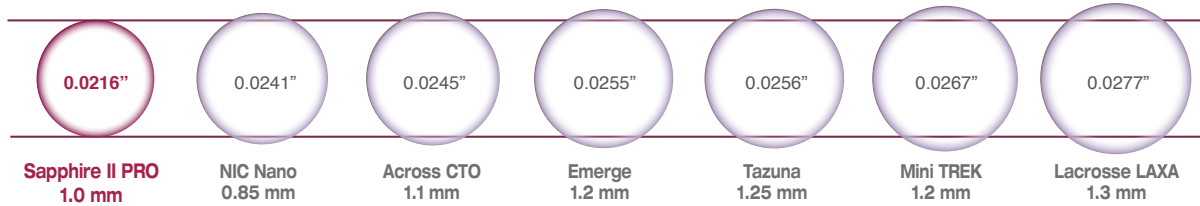

*Data on file, scale depictions

Ordering Information


Balloon Diameter (mm)	Balloon Working Length (mm)					
	5	8	10	12	15	20
1.0	210-053-5UJ	210-083-5UJ	210-103-5UJ	—	210-153-5UJ	—
1.25	212-053-5UJ	212-083-5UJ	212-103-5UJ	—	212-153-5UJ	—
1.5	—	—	215-103-5UJ	215-123-5UJ	215-153-5UJ	215-203-5UJ

Compliance Chart

Pressure (atm)	Balloon Diameter (mm)		
	1.0	1.25	1.5
2	0.94	1.18	1.42
4	0.97	1.22	1.46
6 NOM*	1.00	1.25	1.50
8	1.03	1.28	1.54
10	1.06	1.32	1.58
12	1.09	1.35	1.62
14	1.12	1.38	1.66
16 RBP**	1.15	1.42	1.70
18	1.18	1.45	1.74
20	1.21	1.48	1.78
22	1.24	1.52	1.82

* Nominal Pressure. The nominal in-vitro device specifications do not take into account any lesion resistance.

** Rated Burst Pressure. Do not exceed RBP.

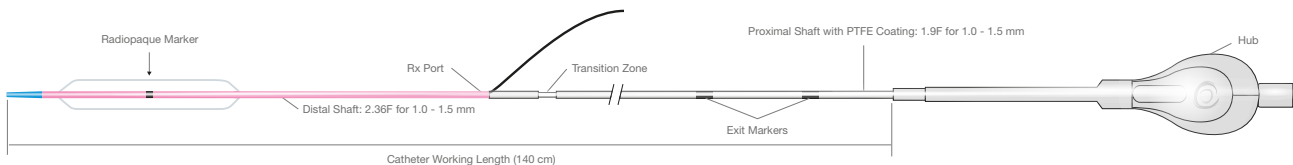

For more information please visit our website at www.OrbusNeich.com or contact us:

CORPORATE HEADQUARTERS

3/F, Building 20E
Hong Kong Science Park
Shatin, N.T., Hong Kong
Phone +852.2802.2288
Fax +852.2507.3532

EMEA HEADQUARTERS

Drs. W. van Royenstraat 5
3871 AN Hoevelaken
The Netherlands
Phone +31.33.254.1150

Toll Free Phone 00800.0254.1150*

* Only for Belgium, Denmark, France, Germany, Ireland, Netherlands, Norway, Sweden and UK

©2020 ORBUSNEICH Medical Corporation or its affiliates. All rights reserved.
The Sapphire II PRO product is now approved in the USA and distributed with USA specific catalog numbers.

G-70-0410 Rev04

SAPPHIRE II PRO and ORBUSNEICH are registered Trademarks of OrbusNeich Medical Corporation.
All data and photos on file. Illustrations are not to be considered as engineering drawings or photos.

 **OrbusNeich[®]**
Pioneers in life-changing technologies